## Evolsin® Nasenpflaster

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## Evolsin® Nasal Strips

The EVOLSIN® NASAL STRIPS are a dermatologically tested medical device for the relief of problems caused by nasal congestion. It helps to ease breathing and reduces snoring.

### PRODUCT INFORMATION:

- Free from active pharmaceutical ingredients
- Widens the nasal cavities
- Facilitates breathing
- Helps reduce snoring
- Hypoallergenic (including the adhesive)
- Free from PVC
- Individually wrapped strips

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### APPLICATION

Apply the strips before going to bed, during the day or before sports activities.

1. In order to achieve an optimal hold of the strip, we recommend that you clean the surface of your nose before applying it and dry it afterwards.
2. Remove the strip from the packaging and take off the protective cover.
3. Place the strip over your nose and press it against the surface of your nose for a few seconds to ensure perfect adhesion.

### WARNINGS

- Do not use on irritated skin.
- Do not use for more than 12 hours a day.
- Remove immediately if skin irritation occurs.
- Do not use on children under 5 years of age.
- For external use only.
- In case of serious incidents, report to the manufacturer and competent authorities.

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EUROSIREL S.p.A.